

# Instruction of P-03 Portable Skin Scrubber




## Product Introduction

Portable skin scrubber, a skin beauty instrument, which is ideal for improve the skin problem. It depends on using the ultrasonic vibration with frequency in 25000t/s to expel the toxin, remove the blackhead, acne and dead skin. This instrument is widely recognized by customers as it is easy to operate and can effectively improve a variety of skin problems. Equipped with the low voltage, this skin scrubber is safe for client to reduce the melanin, sterilize the skin, shrink the skin pore, clean the skin pore deeply, reduce the wrinkle, and whiten the skin.

## Function

1. Remove the aging skin, get rid of the dirt and excess oil from the pore.
2. Refresh the skin and improve skin texture.
3. Deeply clean and clarify the skin without any pain.
4. Dilute pigmentation, ideal for acne skin.
5. Pore Minimizing, skin whitening and shining.
6. Smooth wrinkles, Increase the elasticity of skin.
7. Digital timer for 15min treatment. Instrument stops working automatically when time's up.
8. Portable instrument which is convenient for usage.

## Product Description

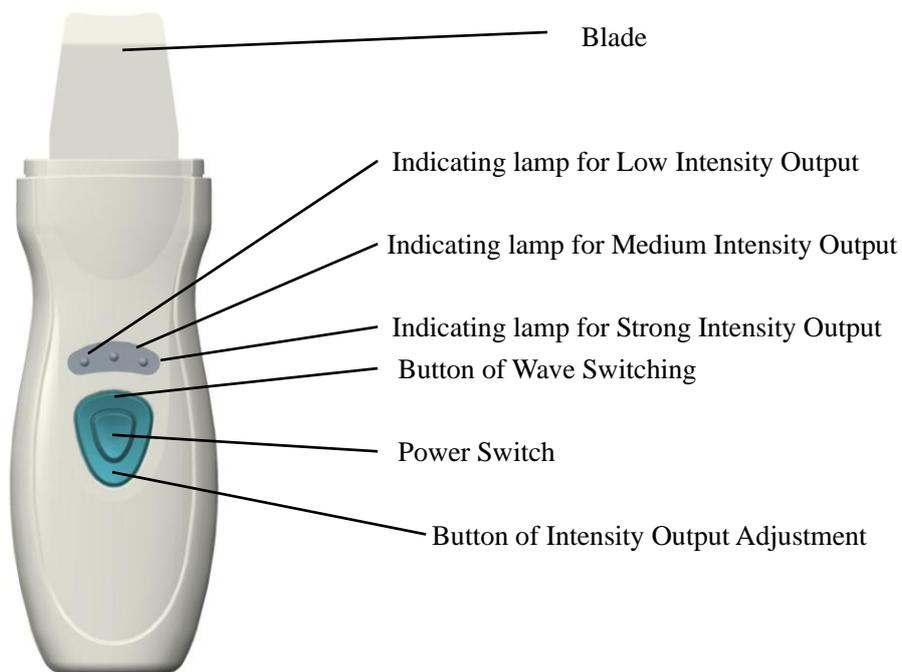

1. Plug the power adapter on the power supply, and then well connect the power adapter and skin scrubber.
2. Switch on the instrument, while indicating lamp lighting. (When you turn on the instrument, the default setting is continuous wave and lower output)
3. Adjust the wave to suit the treatment. There are three output waveforms as option, including continuous wave, pulse wave and combination wave (combination of continuous and pulse). If continuous wave is selected, the indicating lamp will keep on normally. If pulse wave is selected, indicating lamp will keep flickering once 0.25s. And when combination wave is chosen, indicating lamp lights on approx 1s, then flick once 0.25s. Users can switch the wave to suit the treatment as required.
4. Adjust the appropriate output intensity. There are three level output intensity as options, including low, medium and strong level. Default setting is low output.
5. The Default treatment time is 15 min/time. Instrument will beep when time's up, and shut down automatically.


## Operation Guidance

1. Power on the instrument, while it beeps. Switch on the instrument, while indicating lamp lighting and scrubber starts working. (The default setting is continuous wave and lower output.)
2. Adjust the wave and intensity output to suit the treatment. [Combination wave is ideal for the person with sensitive skin.]
3. Clean the treated skin area by applying some lotion or toner evenly on it.
4. Put the scrubber probe on the skin with an inclination angle of 45 degree, see photo as below. Gently move the scrubber probe forward to deeply clean the treated skin. Adjust the output intensity according to the treatment requirement. [Remark: Clean the skin with the probe and clean water, please keep the skin moisturized during the whole treatment.]

**\*\*Use the probe in accordance with the operation shown as illustration A. B. C.**


### A. Remove the acne without pain

The acne in the pores is atomized in a short time by ultrasonic vibration, and then acne is removed out while cleaning the skin.

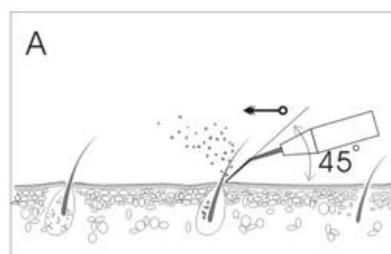


### B. Pox removing

Remove the pox and the residues of cosmetics by the ultrasonic vibration frequency of 25K.

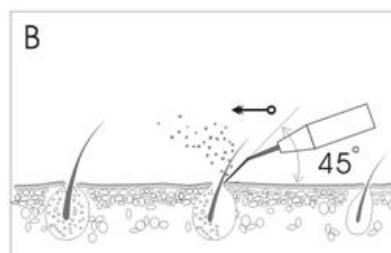

### C. Skin cleaning

The blade is safe for cleaning the dirt on the surface of skin and aging skin. By its effective vibration, the cleaning effect is obvious.

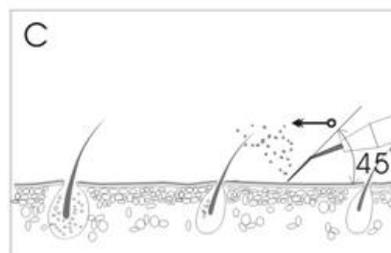

5. Clean the skin with water after treatment.
6. Clean the blade and then sterilize it with alcohol. Well keep it for the next treatment.

## Caution

1. It's ideal for the client with dry and oil skin type. The skin will be refreshed after a long-term treatment.
2. Some water, gel or lotion should be applied on the skin for ultrasonic treatment.
3. When you want to clean the skin or remove the aging skin, continuous wave is the best choice for you. Pulse wave are perfect for massaging skin and smoothing wrinkle.
4. Keep probe clean and the blade should be sterilized before and after treatment.
5. Do not have treatment around the eye and the wound.
6. Do not keep the working probe staying on one position of the skin, so as to avoid the skin be hurt. The probe should keep moving for treatment.
7. Do not continue to use the instrument if ultrasonic probe is damaged or deformed.

8. Please take attention to treat the skin gently, not much force. And please keep the probe slanting with the angle of 45degree on the skin.
9. When the scrubber is working, it's normal that it squeaks. This is because the vibration frequency is similar to the audio frequency.
10. This instrument is unavailable for children and the ones are pregnant or with mental abnormalities.

## **Warning**

1. If the adapter or power cord is damaged, or product fell into the water with careless, please do not continue to use it. It should be returned back to service center for inspection and repair.
2. For indoor use only, and shall not be used outdoors.
3. Do not use product in the environment of high humidity, such as bathroom.
4. Do not use product with other power adapter, as its parameter may not match with our product.
5. Do not use the instrument for those who have skin disease or be sensitive to the ultrasonic stimulation.
6. Power off the instrument if it is not used for a long time.
7. If the power adapter is damaged, please contact manufacturer or service center for replacement. In order to avoid any accident or hazard, repair it personally is prohibited.